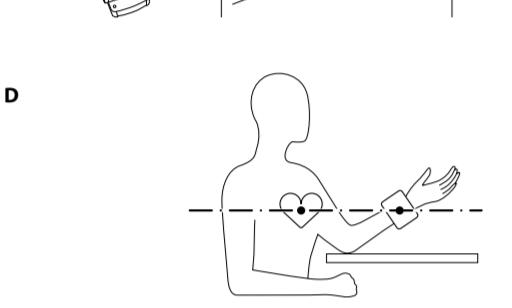
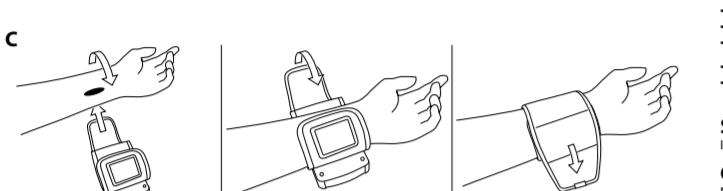
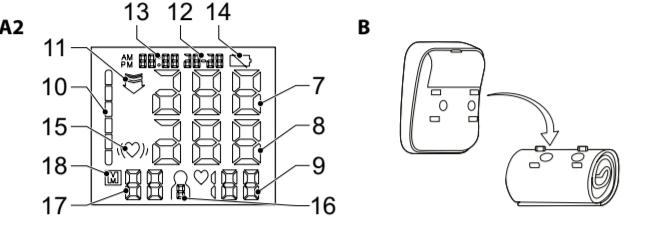
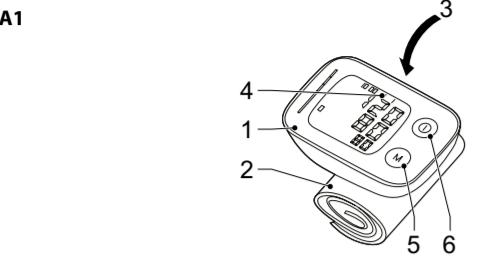
A1

**English - Description (fig. A)**

The blood pressure monitor is a device used to measure a person's blood pressure. The device measures the systolic blood pressure, the diastolic blood pressure and the pulse rate of an adult individual.

1. Monitor	
2. Cuff	
3. Battery compartment	<ul style="list-style-type: none"> <li>Open the battery compartment.</li> <li>Insert the batteries (2x AA) into the battery compartment.</li> <li>Close the battery compartment.</li> </ul>
4. Display	
5. Memory button	<ul style="list-style-type: none"> <li>Press the button to read measurements from the memory.</li> <li>Clock mode: Press the button to set the date and the time.</li> </ul>
6. Start/stop button	
7. Systolic blood pressure indicator	The indicator shows the systolic blood pressure.
8. Diastolic blood pressure indicator	The indicator shows the diastolic blood pressure.
9. Pulse rate indicator	The indicator shows the pulse rate.
10. Classification indicator (blood pressure level)	
11. Inflating/deflating indicator	The indicator shows when the device is inflating (arrow up) or deflating (arrow down).
12. Date indicator	The indicator shows the date.
13. Time indicator	The indicator shows the time.
14. Low-battery indicator	The indicator comes on when the batteries are low.
15. Irregular-heartbeat indicator	The indicator comes on when the device detects an irregular heartbeat.
16. Memory bank indicator	The indicator shows the selected memory bank.
17. Average value indicator	The indicator shows the average value of the all results in the elected memory bank.
18. Memory mode indicator	The indicator comes on when the device is in memory mode.

**Definitions**

- Systolic blood pressure: maximum arterial pressure during contraction of the left ventricle of the heart.
- Diastolic blood pressure: minimum arterial pressure during relaxation and dilation of the ventricles of the heart when the ventricle fill with blood.
- Hypertension: high blood pressure.

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**Safety**

- Read the manual carefully before use. Keep the manual for future reference.
- Only use the device for its intended purposes. Do not use the device for other purposes than described in the manual.
- Do not use the device if any part is damaged or defective. If the device is damaged or defective, replace the device immediately.
- The device is designed for adults. Do not use the device on infants or young children. Before use on older children, consult your physician.
- Do not use the device if you suffer from arrhythmia (irregular heartbeats).
- If you are in doubt about the use of the device or not, consult your physician.
- The readings are only an indication of your blood pressure level. If the device measures a high or low blood pressure level, consult your physician.
- Only use the supplied cuff.
- Measure your blood pressure at approximately the same time each day.
- Before taking a measurement, stay calm and rest for 5 minutes. Relax at least 1 minute between multiple measurements to allow the blood circulation to recover. Too frequent measurements can cause injury due to blood flow interference.
- Do not speak or move during measurement.
- Do not expose the device to water or moisture.
- Do not use the device in areas where interference from other devices can cause inaccurate measurements.
- Do not use the device in a moving vehicle.

**Battery safety**

- Only use the batteries mentioned in the manual.
- Do not use old and new batteries together.
- Do not mix batteries of different types or brands.
- Do not mix batteries of different voltage levels.
- Do not short-circuit or disassemble the batteries.
- Do not expose the batteries to water.
- Do not expose the batteries to fire or excessive heat.
- Do not expose the batteries to cold temperatures. To avoid damage to the product, remove the batteries when leaving the product unattended for longer periods of time.
- If liquid from the batteries comes into contact with skin or clothing, immediately rinse with fresh water.

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**Body posture during measurement (fig. D)**

Refer to the illustration for the correct body posture during measurement

**Measuring the blood pressure**

- To switch on the device, press the start/stop button. The display shows all display items.
- The current memory bank is displayed. Press the memory button to select the memory bank.
- After the start/stop button is pressed, the device enters into clock and time adjustment mode.
- When the device is in standby mode, press the start/stop button and the memory button simultaneously for 3 seconds to enter into clock and time adjustment mode.
- When the device is in the clock and time adjustment mode, press the start/stop button repeatedly to change desired item: Year / Month / Day / Hour / Minutes.
- When the number increases, press the memory button to increase the number. Press and hold the memory button to make the number increase faster.
- The device goes back to standby mode when no button is pressed for 30 seconds in the clock and time adjustment mode.
- When the number flashes, press the start/stop button to confirm the date and time and to switch off the device.

• Refer to the illustration to attach the cuff to the monitor.

• Refer to the illustration to apply the cuff.

• Refer to the illustration for the correct body posture during measurement

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**Cleaning and maintenance****Warning!**

- Do not use cleaning solvents or abrasives.
- Do not clean the inside of the device.
- Do not attempt to repair the device. If the device does not operate correctly, replace it with a new device.
- Clean the outside of the device using a soft, damp cloth.
- If you use the device frequently, disinfect the cuff at least twice a week. Apply ethyl alcohol (75-90%) to a soft cloth, wipe the inner side of the cuff with the cloth and allow the cuff to dry naturally.

**Support**

If you need further help or have comments or suggestions please visit [www.nedis.com/support](http://www.nedis.com/support)

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NEDIS B.V., De Tweeling 28, 5215 MC 's-Hertogenbosch, The Netherlands