Device and controls

Figure 1:


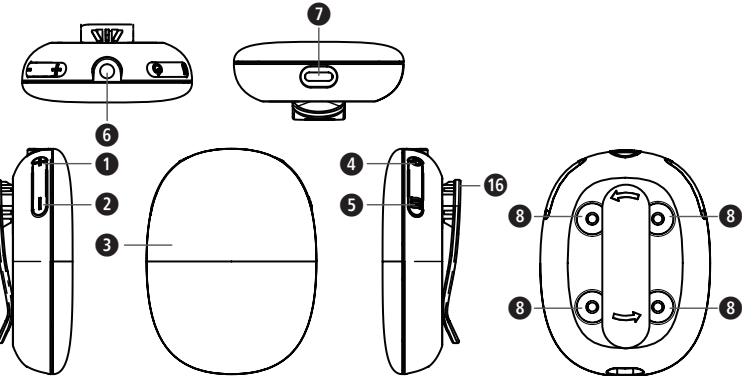
Figure 2:

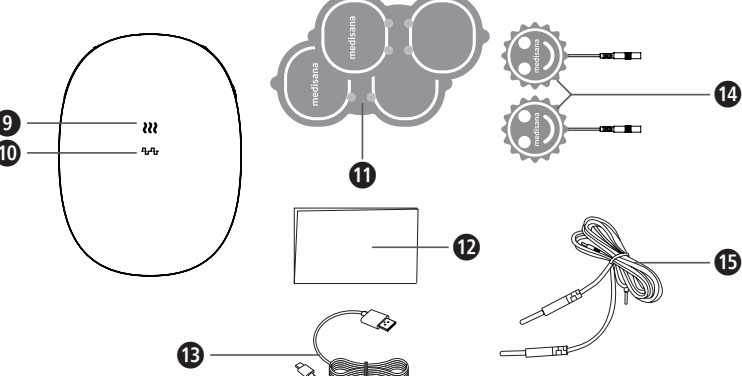Explanation of symbols

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

**WARNING**  
This symbol identifies hazards that may cause serious personal injury or death.

**CAUTION**  
This symbol identifies hazards that may cause minor personal injury, product damage or property damage.

**IP22**  
The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects. Disposal only according to the "WEEE" Waste Electrical and Electronic Equipment EC Directive.

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

Complies with the European Medical Device Regulation (EU) 2017/745. Notified Body is SGS Belgium NV

Catalogue Number

Serial Number

Medical Device

Authorized representative in the European Community / European Union

Unique Device Identifier

Importer

Manufacturer

**medisana GmbH,**  
Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

**Famidoc Technology Co., Ltd.**  
Room 401, 501, 601, 901, Building 6, Zhongtang Tian'an Cyber Park, No. 89, Jinyuan Road, Zhongtang Town, Dongguan City, Guangdong Province, 523220, P.R. China  
Website: www.famidoc.com

**Shanghai International Holding Corp. GmbH**  
(Europe)  
Eiffestraße 80, 20537 Hamburg, GERMANY

**GB IMPORTANT INFORMATION!  
KEEP IN A SAFE PLACE!**

**Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference. If you pass the device on to a third party, this instruction manual must remain with the device.**

**WARNING**

- This therapeutic apparatus is not recommended to use in the following cases:
  - 1) Do not use this device if the patient has any implantable electronic or metal equipment (e.g. heart pacemaker, IUD).
  - 2) Epilepsy.
  - 3) Severe lower extremity arterial circulation problems.
  - 4) Abdominal or inguinal hernia site.
  - 5) Do not use this device if you have a heart disease without consulting a doctor.
  - 6) Being pregnant.
  - 7) Cancer.
  - 8) Having Fever.
  - 9) Do not use therapeutic instruments in high humidity environments, such as bathing or showering in a bathroom bathtub.
  - 10) Do not use after drinking.
  - 11) Do not use Low-frequency therapeutic instruments when using high frequency surgical instruments.
  - 12) Don't use close to the genitals.
  - 13) Recommendation: Do not use low-frequency therapeutic instruments near shortwave or microwave therapeutic equipment, where the output of the low frequency therapeutic instrument used is unstable. Keep a safe distance – at least 1 m – to such devices.
  - 14) The high frequency surgical equipment and stimulator. When connected to a patient, burns may occur at the stimulator electrode and may damage the stimulator.
  - 15) Abnormal feeling of the patient having medical treatment.
  - 16) Patients with cardiac and neurological abnormalities.
  - 17) Patients with body temperature exceeding 38°C.
  - 18) Patients with infectious diseases.
  - 19) Patients who can not express their will.
  - 20) Do not use this product while charging.
- Note:
  - 1) Make sure you are always aware of all warning items.
  - 2) This product should not be used in the throat and mouth. This may result in severe throat muscle spasms and muscle contraction, which will likely block the airway and cause dyspnea.
  - 3) This product should not be used in front of the chest, as the current introduced into the heart can lead to cardiac arrhythmia and cardiac arrest.
  - 4) This product should not be used on the unattended patients, or those suffer from emotional distress, dementia, and with low IQ.
  - 5) Use guidance has been listed in details; any improper use may lead to danger.
  - 6) Do not use this device for any un-diagnosed pain symptoms. Do consult a doctor for diagnosis before use.
  - 7) Electrodes can not be placed in areas that may cause current to pass through the brain (i.e. one-sided electrodes can not be placed on the front and back of the head at the same time, or relative to the left and right sides of the head).
  - 8) Patients suffering from heart disease, cancer or other health-threatening illnesses should only use this device after consulting a doctor.
  - 9) If the intensity of stimulation makes you feel uncomfortable, reduce the intensity of stimulation to a comfortable level. If the problem remains, please contact your doctor.
  - 10) Do not use this equipment in the rooms where aerosol (spray) and pure oxygen are used.
  - 11) Please do not use this equipment near any highly flammable substances, gases or explosives.
  - 12) Please check the cable and related connections are in good condition before each use.
  - 13) Turn off the device before using or removing the electrode pad.
  - 14) This product can only be used with electrode wires and electrodes recommended by the manufacturer.
- Consult your doctor before using the device, especially when:
  - Acute disease, especially in patients suspected or suffering from hypertension, thrombosis or malignant mass.
  - Patients with various skin diseases.
  - Patients with chronic pain disorders (e.g. metabolic disorders) with reduced pain.
  - Patients taking medication.
  - Adverse reactions
  - Skin allergies caused by electrode sheet pad gels and electrode irritation burns are potential adverse reactions. If allergic symptom appears on skin, please discontinue the treatment and consult your doctor.
  - If the intensity of stimulation makes you feel uncomfortable, reduce the intensity of stimulation to a comfortable level. If the problem persists, contact your doctor.

- Do not use this product while charging.
- Illustrations in this manual are effect drawings for reference only. Goods in kind prevail.
- Thanks for using our product. Please pay attention to the instructions and read them carefully before using this product.
- Please ensure the compliance of selected adapters to relevant safety medical level and safety certification requirements (example: IEC 60601-1).
- After connection, it will be charged automatically.

**Working Principle**

TENS, or transcutaneous electrical nerve stimulation, refers to the electrical stimulation of nerves through the skin. TENS is an effective non-pharmacological method for treating different types of pain from a variety of causes. The pain-relieving or pain-suppressing effect is achieved by inhibiting the transfer of pain to nerve fibres and by increasing the secretion of endorphins in the body, as their effect on the central nervous system reduces the sensation of pain. The method is scientifically substantiated and approved as a form of medical treatment.

**Product structure and composition**

- Intensity [+] button
- Intensity [-] button
- Display screen
- On/Off-button
- Heat button
- Connection channel for electrode cable
- Type-C charging interface
- Magnetic fasteners
- Heat icon
- TENS icon
- 2 × TENS & Heating pads
- 1 × this instruction manual
- 1 × Type-C charging cable
- 2 × Electrode pads
- 1 × Electrode wire
- Detachable belt clip

**Notes:**

- This product does not come with an adapter. Please purchase separately.
- If you have any problems with the adapters, consult your dealer at the place of purchase.
- If you notice any transport damage when unpacking, please contact your dealer immediately.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

**Intended use**

The device is intended to temporary pain relief.

- On people
- For external use
- For home and private use
- For muscle pains relief
- For menstrual pain relieve
- For body relaxation

**Contraindications:**

- This device should not be used when cancerous lesions are present in the treatment area.
- Stimulation should not be applied over swollen, infected, inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins, etc.).
- Safety has not been established for the use of therapeutic electrical stimulation during pregnancy.
- Do not use this device if you have an IUD, a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.
- Epilepsy
- Serious arterial circulatory problems in the lower limbs
- Abdominal or inguinal hernia
- Not suitable for use during pregnancy or labor women
- Do not use this device when pain syndromes are undiagnosed. Use only after origin of pain has been diagnosed.
- If you have heart disease without consulting your physician.
- Do not use on scarred areas following surgery for at least 10 months after the operation.
- This device should not be used over poorly enervated areas.
- High fever
- Electrode placements must be avoided that apply current to the carotid sinus region (anterior neck) or transcerebrally (through the head).

**Intended patient population**

For TENS mode: To be used for temporary relief of pain associated with sore and aching muscles due to strain from exercise or normal household work activities. For menstrual pain: Many women experience abdominal cramps and severe period pain before and during their period; and may also suffer from illnesses which cause pain in the lower abdomen, such as endometriosis. These pains are often so strong that women seek immediate pain relief. This device is a self-adhesive TENS device with heat function to provide pain relief for menstrual discomfort and pain caused by endometriosis. The TENS technology used makes it possible to treat pain in a targeted manner. The device features total 20 intensity levels for different demands. It is very a safe and easy way of relieving pain.

**Intended users:** Users must be 18 years or older and be able to control the device and understand the instructions. Professional users are also suitable.  
**Use environment, user and patient group:** Clinics or home use.

**Application**

**Operation steps**

- Connect the port of the Electrode pads ① to the wire terminals.
- Insert the plug of the electrode cable into the channel ② provided in the main device.
- Connect the magnetic clamps from the main unit and the TENS & Heat Pad ③.
- Attach all pads to the treatment area.
- Press «ON/OFF» button ④ and select the program.

**Installation and use [detailed operation description]**

- Place the TENS & heat pad ① and/or the electrode pad ② on the areas to be treated.
- Press the «ON/OFF» button ④ to turn on the device for standby mode (the device emits a "beep" with both icons heating ③ and TENS ④ flashing). If there is no operation for approx. 120 seconds, the device will switch off automatically.
- During standby mode, press [+] button ⑤ to start the TENS function (the device emits a "beep", while the ④ icon stays on and the ③ icon turns off. If heating is needed, press the ③ icon for activation (the buzzer prompts «beep beep» and TENS and heat functions are both active at the same time). If there is no operation for 30 seconds, the device will be locked automatically. Press the [-] button ⑥ to unlock.
- During TENS mode, press [+] button ⑤ to increase intensity - there are 20 levels available. Press the [-] button ⑥ to lower intensity. When intensity has been lowered to level 0, there is no more TENS output and the device will return to standby mode (icons heating ③ and TENS ④ flashing). If there is no operation for approx. 120 seconds, the device will switch off automatically. Remark: There will always be a «beep» sound when pressing buttons ⑤ or ⑥. If you continue to press one of the buttons after max. or min. level has already been reached, the device emits «beep beep beep beep».
- Default treatment time: 30 minutes. When treatment is finished, the device automatically returns to standby mode with both icons heating ③ and TENS ④ flashing.
- In process of TENS treatment, press the ③ icon to start heating (device emits «beep beep»). The default heating temperature is 43°C. To turn it off again, just press the ③ icon again (device emits "beep") and the heating icon will extinguish.
- During heating treatment without TENS, only the ③ icon stays on. If you press the ③ icon, heating will stop and device returns to standby mode.
- To terminate treatment, press «ON/OFF» button ④.
- When the device detects improper electrode pads attachment under the TENS mode, the buzzer will emit << beep beep beep beep>> after 2 seconds and then turn to standby mode. If this happens in the heating mode, the device will not be affected and it will continue the heat function normally until it is completed after 30 minutes. After that device returns to standby mode.
- When charging the product, it will switch from standby/operational mode to charging mode. The heating icon ③ will flash during charging process and will constantly stay on after the device is fully charged.
- When the device has low power, the buzzer will emit <<beep beep beep beep>> and then device will switch off automatically.
- After finishing the treatment, please carefully remove the electrode pads from the skin. Re-paste the protective film over the gel pad for protection.

**Anomalies (Troubleshooting)**

| Issues                             | Possible causes                                                                                                                                  | Solution                                                                                                                                                                                                                |
|------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Unable to boot                     | 1.The battery is empty<br>2. Malfunction                                                                                                         | 1. Charge the device<br>2. Call for after-sales maintenance.                                                                                                                                                            |
| Automatic Stop-ping                | No human body detected                                                                                                                           | 1. The electrode pad was not attached well.<br>2. The skin is too dry. Wet the skin and use it again.<br>3. Electrode pad is abnormal. Replace electrode pad.<br>4. Electrode line is abnormal. Replace electrode line. |
| Key failure                        | The machine is locked                                                                                                                            | Press the [-] button ⑥ to unlock (see operating instructions for details).                                                                                                                                              |
| Weak electrical stimulation signal | Electrode issues:<br>1. Air drying or contamination.<br>2. Position problems or wire problems.<br>3. Wear & tear damage.                         | Replace / Reconnect                                                                                                                                                                                                     |
| Feeling unwell                     | 1. Too much intensity<br>2. Too close distance between electrodes<br>3. Damage to electrodes or conductors<br>4. The electrode area is too small | 1. Reduce strength<br>2. Reorient electrode position<br>3. Replacement<br>4. Replace the electrode pad with an area size of not less than 16 cm (4 × 4 cm).                                                             |
| Stimulus failure                   | Improper placement of electrodes and unknown applications                                                                                        | Replacement of electrodes and follow the operating instructions. Contact your doctor if you cannot solve the problem.                                                                                                   |

**After the treatment is completed / Cleaning**

Clean the electrode pads and the main unit carefully if they have become dirty or soiled. Therefore, wipe the surfaces with a soft, slightly moistened (use water or a soft, neutral detergent) cloth. Let it dry thoroughly before using it again.

**Technical Specifications**

|                         |                                                                                                                              |
|-------------------------|------------------------------------------------------------------------------------------------------------------------------|
| Name and model:         | <b>medisana</b> Menstrual Pain Reliever with TENS & Heat <b>TT 250</b> (model FDES106)                                       |
| Charging time:          | ≤ 2 hours                                                                                                                    |
| Output current:         | max. 70 mA                                                                                                                   |
| Output voltage (peak):  | max. 35V ±20%                                                                                                                |
| Power supply:           | 3.7V lithium battery (5V/1A adapter)                                                                                         |
| Load resistance:        | 500Ω                                                                                                                         |
| Product service life    | 5 years                                                                                                                      |
| Operating environment:  | Temperature: 41°F – 104°F / 5°C – 40°C,<br>Humidity: 30%RH – 85%RH, no condensation,<br>Atmospheric pressure: 70kPa – 106kPa |
| Storage environment:    | Temperature: 14°F – 122°F / -10°C – 50°C,<br>Humidity: 15%RH – 90%RH,<br>Atmospheric pressure: 70kPa – 106kPa                |
| Type of waveform:       | TENS / Pulse type: Two-way symmetrical square wave                                                                           |
| Number of channels:     | 2 channels                                                                                                                   |
| Intensity:              | 0–20 (adjustable)                                                                                                            |
| Output frequency:       | 8–100Hz ±10%                                                                                                                 |
| Output pulse width:     | 75–250µs ±10%                                                                                                                |
| Treatment time:         | 30 minutes                                                                                                                   |
| Electric shock:         | Class II equipment                                                                                                           |
| Applicable part:        | BF Application section                                                                                                       |
| Water protection grade: | IP 22                                                                                                                        |
| Heating temperature:    | 43°C ±2°C                                                                                                                    |
| Size of heating pads:   | 181 × 86,33 mm ±0,5 mm                                                                                                       |
| Dimensions (L × W × H): | approx. 52 × 69 × 24 mm                                                                                                      |
| Weight:                 | approx. 41 g                                                                                                                 |
| Article number:         | 88352                                                                                                                        |
| EAN number:             | 4015588 88352 1                                                                                                              |

**Replacement parts (available separately – not part of the TT 250 delivery):**

- Set with 2 x TENS & heat pads, art. 88353 / EAN 4015588 88353 8
- Set with 4 electrode pads, art. 88354 / EAN 401558888354 5
- Electrode cable, Art. 88355 / EAN 4015588 88355 2

**Disposal — Waste Electrical and Electronic Equipment (WEEE)**

This product is subject to European Directive 2012/19/EU on waste electrical and electronic equipment and is marked accordingly. Never dispose of electronic devices with household waste. Please seek out information about the local regulations with regard to the correct disposal of electrical and electronic products. Correct disposal helps to protect the environment and human health.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your local authority or your supplier for information about disposal.

**Warranty and repair terms**

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Applied parts which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

The complete instructions for use are available for download at <https://docs.medisana.com/88352>.

Information about service can be found here: <https://www.medisana.com/servicepartners>

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